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## Soft Collar

TC 01

### Reduces pain, treats neck spasm, provides support for the head

Soft collars are used in minor neck injuries and cramps. By reducing the range of motion of the neck and head, this necklace makes the muscles, tendons and ligaments in the neck vertebrae rest and are not under tension. Also, after tying the collar, the weight of the head is supported by the collar and the pressure load is removed from the neck vertebrae. As the pressure load is removed from the vertebrae, muscles, tendons and ligaments begin to regenerate.

#### Features and Benefits:

- Has a place to place the chin
- Not causing skin sensitivity
- No pressure on the clavicle and chin
- Easy installation

#### How to Use:

Place the collar around your neck so that the chin strap is under your chin. Then tie the collar.

## Semi-Rigid Collar

TC 05

### Reduces pain, treats neck spasm, provides support for the head

Semi-rigid collars are used in relatively sensitive neck injuries. Due to semi-rigid structure of the foam used in the collar, This type of collar gives more stability to the neck and by reducing the range of motion of the neck and head, it causes themuscles, tendons and ligaments in the area of the vertebrae to get rest. Also, after tying the collar, the weight of the head is supported by the collar and the pressure load is removed from the neck vertebrae.


#### Features and Benefits:

- Neck Spasm
- Injuries to the neck due to accidents
- Recovery period after neck vertebrae surgery

• Relatively severe arthritis of the neck

#### How to Use:

Place the collar around your neck so that the chin strap is under your chin. Then tie the collar.

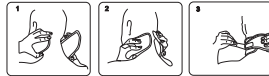## Rigid Collar

TC 10

### Fully Stabilizes the Neck and Prevents Neck Movements, Supports the Neck

The hard neck brace is used to stabilize and fully support the neck in disc and cervical vertebrae replacement surgeries. After tying the necklace, the weight of the head is supported by the necklace and the pressure load is removed from the neck vertebrae. By removing most of the weight of the head from the neck, as well as restricting movement in the neck, the vertebral bones begin to regenerate and stabilize the implant and artificial cervical disc in the vertebral bone.

#### Features and Benefits:

- It has a hard and solid structure
- Limit movement in the head
- Carrying more weight of the head by the necklace
- It has a place to place the chin
- Not causing skin sensitivity
- No pressure on the clavicle and chin
- Easy installation

#### How to Use:

Place the collar around your neck so that the chin strap is under your chin. Then tie the collar.

## Posture Corrector

TC 33

### Two splints to apply controlled pressure, pressure on the spine, preventing back pain, neat appearance.

Hunchback is one of the abnormalities of the thoracic region of the spine, which causes a person to bend forward and create an inappropriate appearance. In


addition, hunchback causes severe back pain and if not treated, causes permanent deformity in the spine. For this reason, humpbacks with splints, by creating controlled pressure in the thoracic region of the spine, correct this deformity and create a proper appearance.

#### Features and Benefits:

- No pressure on the chest and abdomen
- No pressure in the armpit and shoulder
- Not restricting the range of motion of the spine
- Reduction of pain caused by back hump
- Increasing the self confidence
- Strengthens muscles

#### How to Use:

First, fasten the short straps under the chest. Then pass the long straps over the shoulder and reach from the armpit to the back. Then cross each long strap across the back and on top of each other and at the end tie them in front of the body and under the chest.

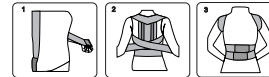#### Maintenance:

Avoid washing the brace with hot water. Avoid washing the brace with washing machine. Wash with soap. Avoid pressing and ironing the brace.

#### Warning:

Consult your doctor for information on the method and duration of use. Avoid tightening the knee brace too tightly. Avoid applying direct and indirect heat on it. Prevent joint use between people with skin disease.

# User Manual

## Product Ref:

TC 01  
TC 05  
TC 10  
TC 33

**Our Goal** Specialized products for the care of joints and bones, while preventing damage, help in the treatment and recovery stages of the consumer's mobility and lead to maintaining health and increasing the quality of life. OTTESY, is Product for All Walks of Life

☎ 021-4422-3675 ✉ info@otessy.com

📄 otessyofficial 🌐 www.otessy.com

Paper Num: T-R-02-121  
Date of review: 1402/12/07  
Review Num: 1