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Hallux Valgus Splint-Big Toe TF 10 Correction of hallux valgus deformity, with a movable joint

Big toe hump is one of the disorders that occurs in the big toe in both genetic and acquired ways. Its acquired form has a direct relationship with inappropriate shoes and incorrect walking, which causes the joint of the big toe to have a hump (Hallux Valgus).

The hallux valgus treatment kit, by having a joint at the point of contact of the thumb, by applying tension on the thumb causes the finger to be pulled outward and corrects the problem. In addition to correcting the problem, the hallux valgus therapy kit allows the thumb to move due to its joint in the thumb area, and its mobility prevents the thumb joint from becoming dry.

Features and Benefits:

- Has a movable joint
- It has two separate straps to fit on the splint on the fingers
- It has a soft pad to reduce pressure and reduce pain in the knuckle area

How to Use:

Insert your foot into the larger strap so that the strap is under the arch of the foot. Then open the finger straps and close them around the thumb and the side finger. Note that the joint is adjacent to the thumb joint. Then close the main straps under and on the leg.

Hallux Valgus Splint-Corrector with Neoprene Straps TF 20 Correction of hallux valgus, with aluminum splint

Big toe hump is one of the disorders that occurs in the big toe in both genetic and acquired ways. Its acquired form has a direct relationship with inappropriate shoes and incorrect walking, which causes the joint of the big toe to have a hump (Hallux Valgus).


The hallux valgus correction splint with a fixator strap is a splint designed to correct severe hallux valgus deformity. The fact that this product has a splint makes it help to correct the alignment and treat the deformity by putting a lot of pressure on the joint of the thumb. Due to having an adjustable strap, this product can put the thumb in its natural alignment and correct its abnormality.

Features and Benefits:

- It has an aluminum splint in the place of contact with the thumb
- Applying tensile pressure to the outside by the splint to correct the deformity
- With a stabilizing strap

How to Use:

Place the splint on the outer side of the thumb and tie the thumb strap around the thumb. Pass the long strap through the back of the leg and fasten it.

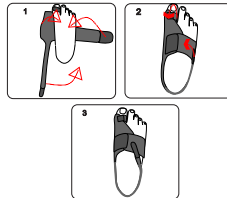Arm Sling - Double Pouch TS 01 New Design, Double Pouch Mechanism,

In fractures of the forearm and wrist bones, it is necessary to remove the fixed hand and the weight from it so that the bone can heal. For this reason, one of the ways to hold and suspend the hand is to use a hand pendant. Otessy Arm Sling has two elbow and wrist support bags, which makes the hand suspended and the weight is removed from it. The pendant around the neck of this product has a soft pad that does not cause discomfort and damage to the neck and creates comfort in the neck.

Features and Benefits:

- Adjustable size
- Ergonomic design

- Has two holding bags
- Soft neck pad
- Beautiful and attractive design
- Durable and strong

How to Use:

Place the padded neck area behind your neck so that the pendant hangs from your neck. Then put the first bag under the armpit of the injured hand and the second bag over the chest and put your hands in the bags.

Maintenance:

- Avoid washing the brace with hot water.
- Avoid washing the brace with washing machine.
- Wash with soap.
- Avoid pressing and ironing the brace.

Warning:

- Consult your doctor for information on the method and duration of use.
- Avoid tightening the knee brace too tightly.
- Avoid applying direct and indirect heat on it.
- Prevent joint use between people with skin disease.

OTESSY[®]
Products for all walks of life

User Manual

Product Ref:

TF 10
TF 20
TS 01

Our Goal Specialized products for the care of joints and bones, while preventing damage, help in the treatment and recovery stages of the consumer's mobility and lead to maintaining health and increasing the quality of life. OTESSY, is Product for All Walks of Life.

☎ 021-4422-3675 ✉ info@otessy.com
📄 otessyofficial 🌐 www.otessy.com

Paper Num: T-R-02-122
Date of review: 1402/12/07
Review Num: 1